

Daily cleaning of frequently touched surfaces with PCS Four Sided Wipes and 50 ppm of hypochlorous acid/ neutral pH sodium hypochlorite.

Dilute PCS 1000 Plus by adding 40 mL to 1 L of water to make up 50 ppm cleaning solution in labelled workplace bottle with PCS-TRG-12 HYPOCHLOROUS ACID AND SODIUM HYPOCHLORITE COMPATIBLE sprayer.
 OR
 Dilute 25 mL of NPH 2000 per 1 L of water through portable dispenser. Make up 50 ppm cleaning solution in labelled workplace bottle with PCS-TRG-12 HYPOCHLOROUS ACID AND SODIUM HYPOCHLORITE COMPATIBLE Sprayer.

Squirt on to PCS Four Sided Wipe and wipe surfaces, squirt additional cleaning solution on surfaces to keep PCS Four Sided Wipe moist and effective at removing soil over large areas.

To clean and decontaminate iPhone, tablets and portable electronic medical equipment.

Spray 50 ppm solution on PCS Four Sided Wipe to dampen cloth.

Wipe surfaces with damp cloth and immediately wipe surfaces dry with Toraysee™ cloth.

Broad Spectrum Virucidal Hospital Disinfecting with PCS Four Sided Wipes and PCS 1000 Plus Oxidizing Disinfectant Cleaner.

Squirt PCS 1000 Plus on PCS Four-Sided Wipe and continue to apply additional PCS 1000 Plus so all surfaces wiped remain damp for contact time and receive fresh active ingredient. Using this process allows you to apply disinfecting solution effectively over a much larger area than you can effectively wipe with a pre moistened canister wiper.

