PCS promotes the use of best practice with new disinfectant wipes packaging

PCS 1000 Oxidizing Disinfectant/Disinfectant Cleaner Wipes are now packaged in a convenient pouch containing one container with 0.1% of sodium hypochlorite and a roll of 70 dry wipes. This way of packaging offers a 2 year shelf life in sealed unopened containers and is designed to be safer for staff by eliminating the need to touch pre-moistened wipes while threading the container.

New packaging and process

To make pre-moistened wipes.
Remove container and dry wipes from plastic pouch.

Place container on a flat surface and secure with one hand. Pry top of container towards you with the other hand to remove. OR Place container on a flat surface and secure with two hands. Push upwards with both thumbs to remove the top from the container.

Once open peel the protective seal off slowly.

Slowly place roll of dry wipes into container.

Thread wipe through lid and snap lid back onto container.

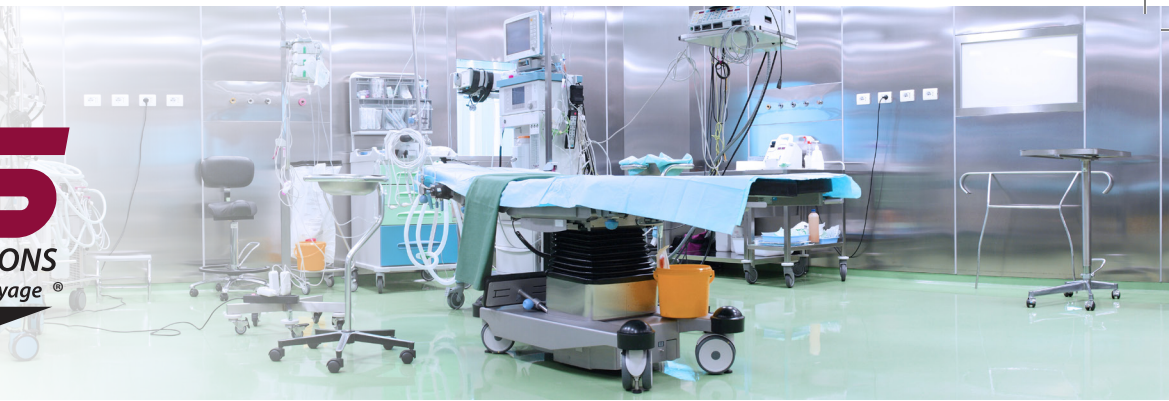
Dispense wipe from container, use to clean and disinfect surfaces. Discard in trash.

Keep lid closed.

PCS Activation Stickers

Allows for easy and convenient identification when containers are put into service.

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