

stryker®



IT'S ALL ABOUT THE PROCESS



PCS PROCESS FOR CLEANING AND DISINFECTING
COMPLIANT WITH STRYKER RECOMMENDATIONS

www.processcleaningsolutions.com

1-877-745-7277

PCS process for cleaning and disinfecting compliant with Stryker recommendations for patient handling equipment beds, stretchers and mattresses.



EQUIPMENT BED RAILING AND OTHER SURFACES NOT MATTRESSES

1. Using PCS hypochlorite/bleach wiper wipe over surfaces to be cleaned.
2. Take a PCS microfibre cloth soaked in warm water and wipe surfaces to rinse.
3. Take a dry PCS microfibre cloth and dry surfaces.



MATTRESS CLEANING

1. Take a PCS microfibre cloth saturated with diluted MicroClean wipe Mattress cover thoroughly.
2. Take a dry PCS microfibre cloth and dry surface.
3. Apply PCS hypochlorite disinfectant with PCS wiper system.
4. Take a PCS Microfibre cloth saturated in water only and wipe/ rinse surfaces.
5. Allow to air dry

Choose a ready to use PCS bleach wiper or disinfecting wipe or create your wipers from a PCS ready to use Hypochlorite disinfectant or concentrate.

PCS has registered hospital grade hypochlorite disinfectants at use concentrations of 225, 250 , 1000, 2500 and 5000 parts per million of hypochlorite.